Sensors

English

Thank you for purchasing a SleepSense® sensor.

Description

Using a piezo-electric crystal, the respiration effort sensor converts chest or abdominal respiration motions to a small analog voltage that provides a clear, reliable indication of respiration waveforms. The micro-effort sensor is especially small and sensitive to higher frequency respiration movements, and was designed for monitoring respiration movements of infants and preterm babies. The sensor does not require a battery or interface; it is easily activated by plugging the connector/s into any AC (low level) channel on the recorder.

Indications for use

SleepSense® sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Sensor

The patient should lie on his/her back for optimal positioning of the sensor.

Positioning the Sensor

Apply the sensor on the chest wall halfway between the nipple and the midline. Secure the sensor in place with a piece of adhesive tape placed from side to side and protruding at least 2 cm from each side.

Cleaning the Sensor

- Wipe the sensor and cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- Do not soak the sensor in disinfectants.

Technical Specifications:

Cable:	White insulation tinsel wire
Sensor:	Square flexible TPE disc
Recommended filter settings:	High pass: 0.1 Hz / Low pass: 70 Hz, (or as recommended in the system's instructions).
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

!

Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrant the Micro Effort Sensor to be free of defects in materials and workmanship for a period of six months from the date purchased. The sole liability of SLP and our Distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our Distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This Warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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