Descrizione

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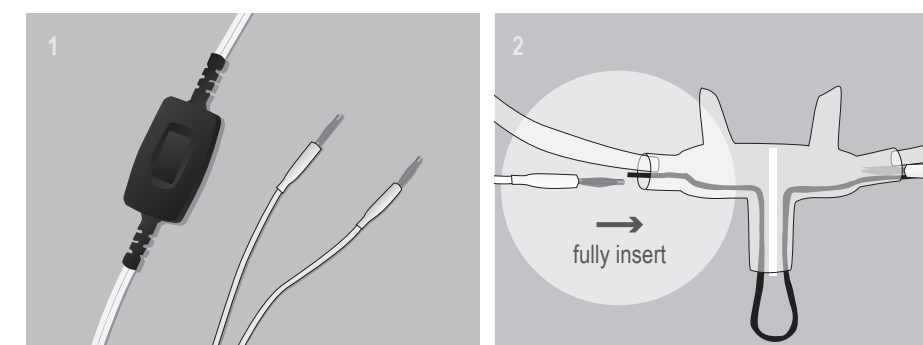
Thank you for purchasing a SleepSense® sensor.

Description

The ThermoCan™ Interface Cable figure 1 is used to connect the ThermoCan™ disposable thermal + pressure cannula figure 2 to the PSG system.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system.

Applying the ThermoCan™ Interface Cable

- 1. Connect the Interface Cable to the PGS system.
2. Slide the Interface Cable's golden pins into the holes provided on the sides of the cannula nose piece.

Cleaning the Interface Cable

- Wipe the cable and golden pins with a non-corrosive (to plastic) cleanser to clean before use.
• Make sure the complete sensor assembly is thoroughly dry before reusing it.

Technical Specifications:

Table with 2 columns: Specification (Cable, Sensor, Recommended Filter Settings, Signal Output, Operating conditions, Storage temperature, Operating and storage humidity) and Value.

Warnings and Precautions

- Caution: US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
• SleepSense sensors are for professional use only.

Limited Warranty
SLP warrants the ThermoCan™ Interface Cable to be free of defects in materials and workmanship for a period of d of twelve months from the date purchased.

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