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SleepSense®
The Sensor Solution Company

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Belts

English

Thank you for purchasing a SleepSense® sensor.

Description

The inductance of the inductive belts is exactly proportional to the cross section area of the band's closed loop around the body. It therefore changes as the patient breathes. The electronics in the system or interface box measure these inductance changes to generate a signal that is the most accurate and reliable replication of respiration waveforms. Inductive belts easily connect to any system compatible with inductive effort sensors through a dedicated SleepSense interface cable.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Positioning and Connecting the Belt (Using a SleepSense Interface Cable)

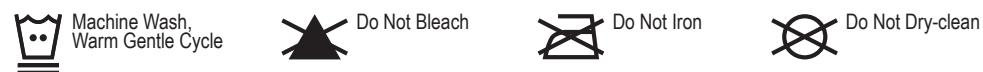
- 1. The patient should lie on his/her back for optimal positioning of the belt.
2. Select the proper size belt for the patient.
3. Strap the sensor around the patient's abdomen or chest over clothing.
4. Attach the first wire to the snap that is on the same side of the belt as the Velcro® patch.
5. While holding this end to the left of the patient's centerline, wrap the belt around the patient's chest or abdomen. For smaller patients move the starting point further to the left.
6. Pull the other end over the first end and secure to the belt on the right side of the centerline. The belt should be snug but not too tight, (see image).
7. Attach the second wire to the exposed snap. Secure both wires about 3" away from the connection.

- Check that clear, strong signals are being transmitted before leaving the patient.
• When removing the interface cable do not pull on the wires. Remove from the snap.

Attention! Using two chest or two abdomen interface cables on the same patient may cause interference.

Cleaning the Belt

- The belt are machine washable.
• Make sure the belt is thoroughly dry before re-use.
• Do not soak the belt in disinfectants.

Operating/Storage

Table with 2 columns: Item #, 9003/9002 (different lengths). Rows include Belt, Operating conditions, Storage temperature, and Operating and storage humidity.

Warnings and Precautions section with a warning icon and a list of caution points regarding US Federal Law and professional use.

Limited Warranty
SLP warrants the Semi Reusable Inductive Belts to be free of defects in materials and workmanship for a period of 60 days from the date purchased.

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