
Sensors

Item Nq1XX-B - var 2 1 [Side A]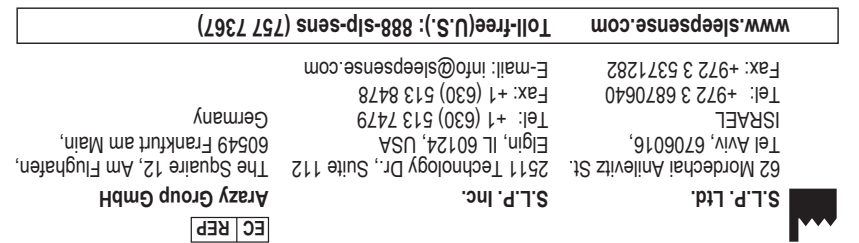

Specifications are subject to change without notice
Les spécifications sont sujettes à modification sans préavis
Especificaciones sujetas a cambio sin previo aviso
Le specificazioni sono soggette a modifica senza preavviso
Die Spezifikationen sind ohne vorherige Meldung Änderungen unterworfen

Induktive Kupplungsschnalle

Sensors

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Thank you for purchasing a SleepSense® sensor.

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Description

The RIP interface buckles are intended to connect between the SleepSense Disposable Inductive Bands and systems with a built-in Inductive channel.

Positioning the Sensor

- Select the proper size band for the patient.
- Follow attached drawing indicating how to place the bands over the patient’s Chest / Abdomen.
- Check that clear, strong signals are being transmitted before leaving the patient.
- Before removing the sensor bands, unlock the handle. Do not pull while handle is locked.

Cleaning the Interface Buckle

- Wipe the patient exposed parts of the interface buckle with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the interface buckle is thoroughly dry before reusing it.
- Do not soak the Interface Buckle in disinfectants.

Technical Specifications:

Name:	Interface Buckles
Sensor:	Inductive Effort Sensor Interface Buckle
Recommended filter settings:	High pass: 0.1 Hz / Low pass: 40 Hz or as recommended in the system’s instructions
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

!

Warnings and Precautions

- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors are for professional use only.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrants the Inductive Interface Buckle to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our Distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our Distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This Warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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